

 **ACCUREX**

# ACCUSAFE Touch

Safety Lancets



Serving Healthcare Professional's  
Right and Patient's Need to  
**Safety & Comfort**

 **point of care**  
*Test and Treat*

## KEY HIGHLIGHTS



### Safe

- ▶ **Auto-retractable** : Prevents accidental needlestick injuries (NSI) before and after use
- ▶ **Single-use and Sterile**: Eliminates cross infection
- ▶ **Internationally Certified**: CE, USFDA, TUV, OSHA-Compliant and meets CDC Guidelines



### Simple

- ▶ **Quick**: Preset; no loading device needed
- ▶ **Innovative**: Patented ergonomic design and touch-activated



### Comfortable

- ▶ **Virtually Painfree**: Ultrafine smooth needle; siliconised and electropolished
- ▶ **Consistent**: Fixed Needle Gauge (28G) and controlled penetration depth (1.8mm) for consistent blood volumes



### Cost Effective

- ▶ Minimal training required
- ▶ Pack sizes available: 100's

\* OSHA - Occupational Safety and Health Administration requirements mandating Safety - Engineered needle devices  
CE - European Conformity  
TÜV Technical Inspection Association Certified  
CDC - Centre for Disease Control  
NSI - Needle stick Injury

## How to use

### 1. Decap



Remove the protective cap from the safety lancet.

### 2. Touch



Place the white end of safety lancet at the test site and press it firmly against the test site to activate the device.

### 3. Collect



Gently massage the finger in the direction of test site to obtain the required blood volume.

Dispose of the used safety lancet in sharps disposal container.



**Biomedical Pvt. Ltd.**

**Head Office:** 212, Udyog Mandir No. 1, 7/C, Bhagoji Keer Marg, Mahim, Mumbai - 400 016 INDIA. Tel: + 91(22) 67446744

**Factory:** G-54, MIDC Tarapur, Boisar, Thane - 401 506. INDIA. Tel: +91(2525) 645911/17

**Website:** www.accurex.net | **Email:** info@accurex.net | **Call us Toll Free on : 1800 209 8456**